

How should you take care of your implant-supported dental prosthesis?


Crown or bridges supported by dental implants require the same care and treatment as natural teeth. They should be brushed and flossed regularly in order to keep clean and plaque free.



personal touch
professional approach

www.adin-implants.com

MB010-06 10.24

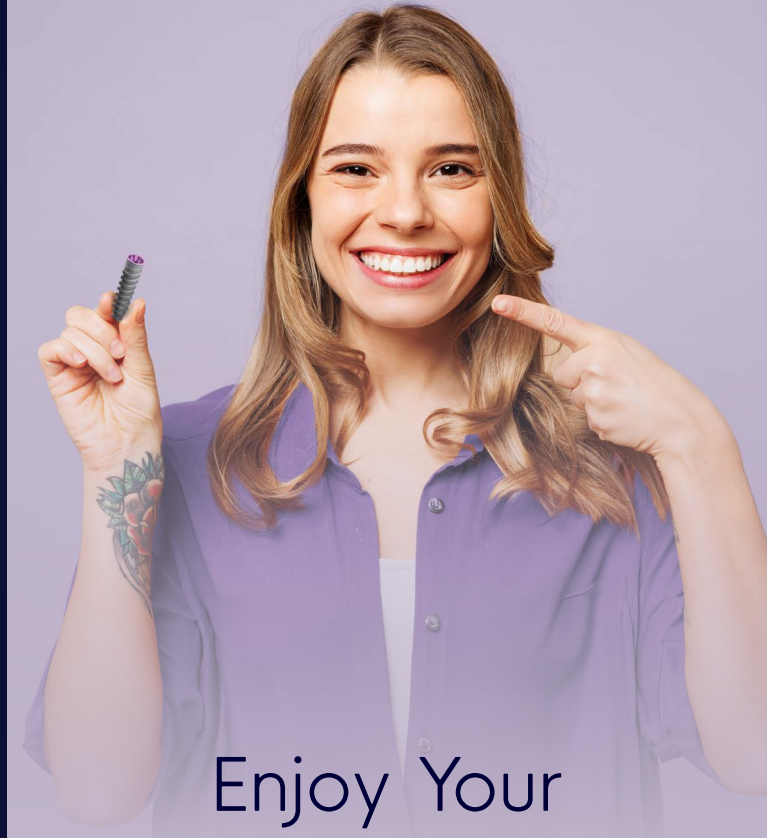

Enjoy Your Smile Again

All you need to know
about dental implants




What are dental implants?

Dental implants are innovative, screw-like posts that expertly designed to replace the roots of missing teeth. Surgically inserted into the jawbone beneath the gums, they create a strong foundation for dental prosthetics, including crowns, bridges, and dentures. Experience the benefits of improved oral health, enhanced chewing function, and a radiant, natural smile. You can reclaim your confidence with dental implants.

Why replace lost teeth with implants?

Dental implants are permanent, reliable and designed to last for many years, keep your jaw bone volume and quality and prevent bone loss and gum recession.

Fixed crown or bridges supported by implants look, feel and function just like natural teeth. They restore your ability to chew. They restore your ability to chew, improve your speaking confidence, enhance your aesthetic appearance and restore your natural smile.

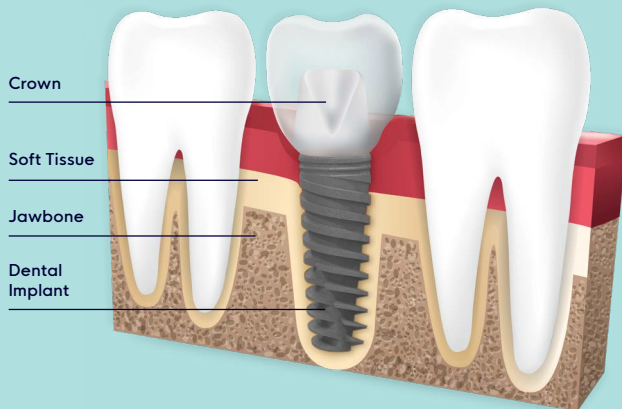


What are the treatment steps?

1. A qualified dentist thoroughly examines your teeth and determines the implant treatment procedure that is right for you.
2. Your implants are inserted via a surgical procedure. The procedure is not painful since it is performed under local anesthesia. Any pain or discomfort after the surgery can be managed with analgesic drugs.
3. On most occasions, implants require a healing period of 3 to 6 months before prosthesis placement. Sometimes, depending on surgical conditions, a prosthesis may be placed on the same day, in what is generally called immediate function.
4. It is at the dentist's discretion when the adequate prosthesis can be placed. The prostheses can be fixed (like a crown or bridge) or removable (like dentures).




What are the different types of restorative options?

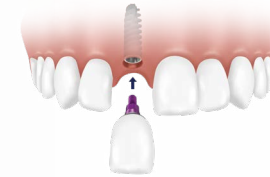


Single Tooth Restoration

An implant is inserted to replace the root of the missing tooth to support crown fixture.

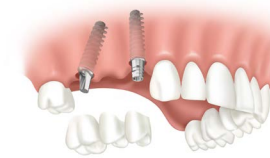

Multiple Teeth Restoration

Implants are inserted where the teeth are missing, in order to support several crowns or a bridge.

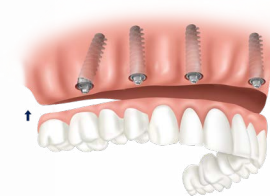

Full Mouth Restoration

A fixed denture supported by four or more implants will provide you with permanent teeth, restore your natural smile and enhance your self-confidence.